G I B A U D

ACTION V DOUBLE

6675/6674

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EN | Instructions for Use

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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

Lower back conditions that may benefit from gross motion restriction, such as:

- Muscular low back pain.
- Lumbosacral strains.
- Chronic low back pain.
- Sciatica.

PERFORMANCE

The device provides lumbar spine gross motion restriction.

The device reduces stress on the low back in flexion and extension.

CONTRAINDICATION

Must not be used by individuals for whom compression is contraindicated.

WARNINGS • PRECAUTIONS OF USE

Patients suffering from pelvic floor weakness should consult with their medical professional before using the orthosis.

This product contains Naturel Rubber Latex which may cause allergic reactions.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Gradual withdrawal possibly combined with functional physiotherapy.

FEATURES OF THE DEVICE

The device is a lumbar stretchable belt featuring four V-shaped posterior stays.

Ergonomic finger loops facilitate installation.

SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

During testing, the fitter must shape the V-shaped stays of the brace to adjust it to your form.

- Wear over an undergarment or light clothing.

1 - Position the bottom part of the belt at the top of the gluteal cleft.

2 - Center the dorsal stays on either side of the spine.

The Gibaud logo must be on the right side.

3 - Adjust the tightening according to pain intensity and activity so that it is firm but comfortable.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Store the brace away from sources of extreme heat (including radiators and tumble dryers). Hand wash in warm soapy water (maximum 40°C), rinse thoroughly. Wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

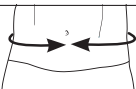
The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

The device is available in two heights: 21 cm (6675) & 26 cm (6674).

The size is to choose in accordance with the waist circumference, measured 2 cm above the iliac crests (in cm)

	0	1	2	3	4	5
	70-80	80-90	90-100	100-110	110-125	125-140

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GIBAUD S.A.S.U