CL1

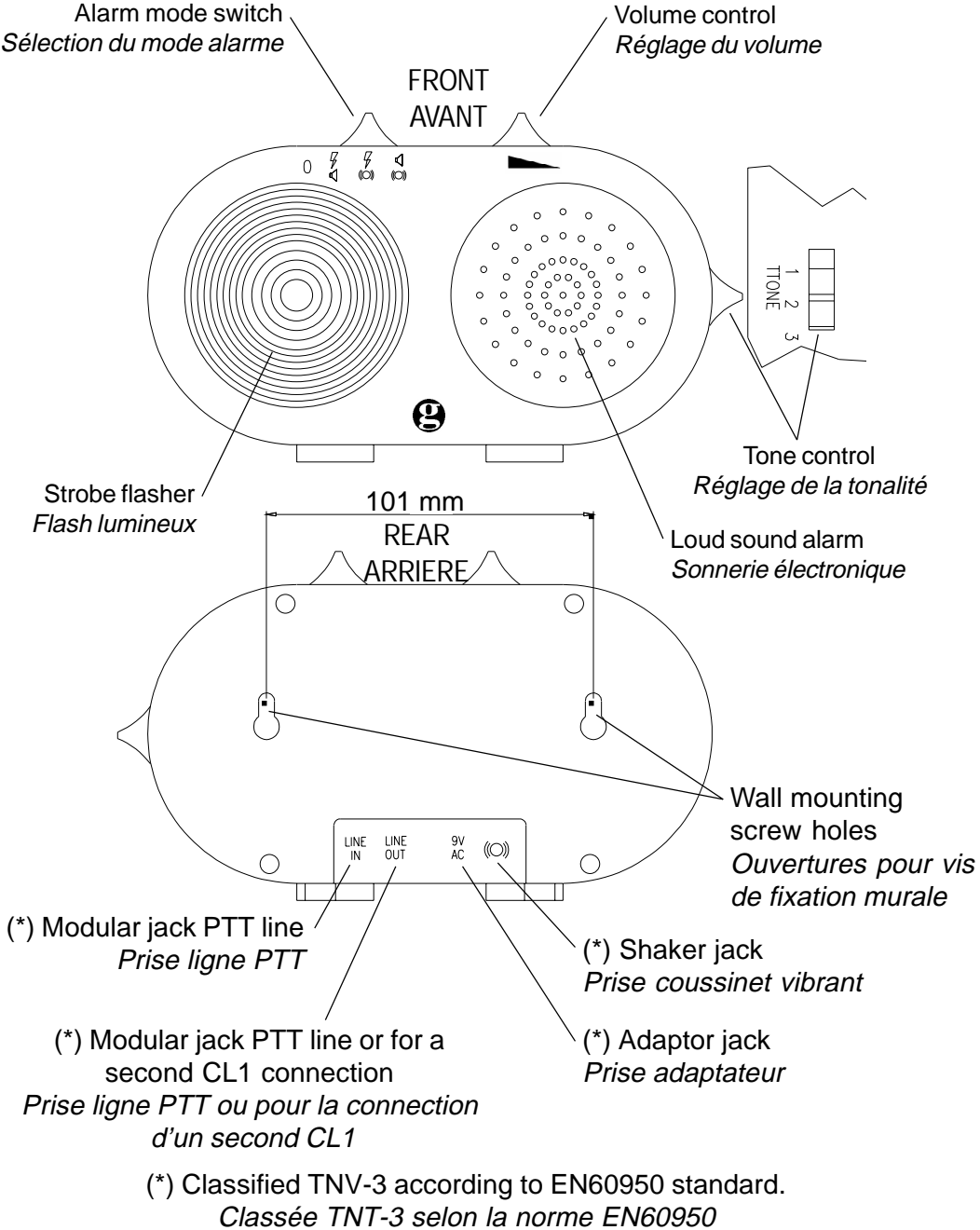
Telephone ring amplifier and flasher
User guide

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
DESCRIPTION

INSTALLATION (See drawing)

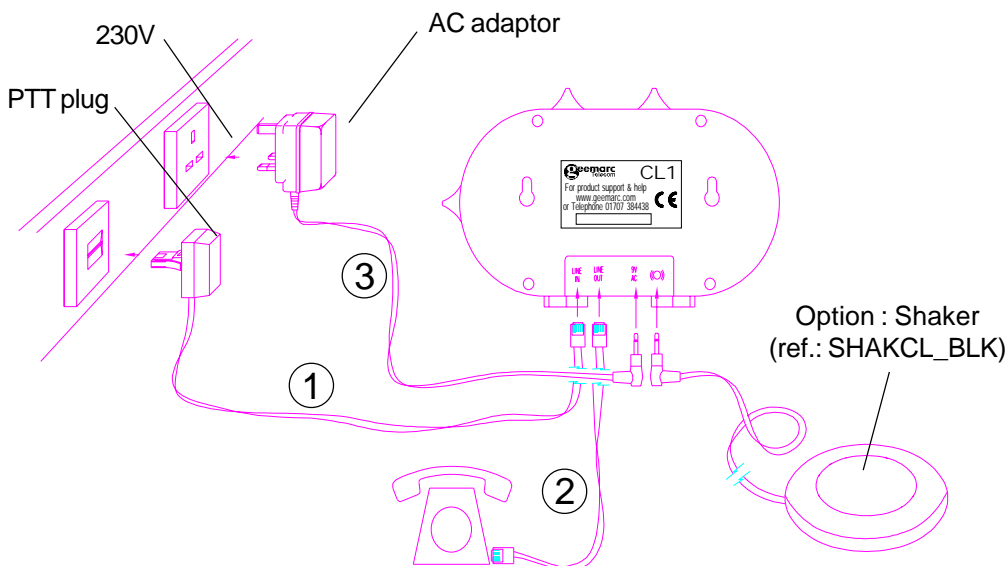- 1 - Connect the PTT line cord.
- 2 - Connect the telephone line cord.
- 3 - Connect the adaptor(*).

(*) Classified "hazardous voltage" according to EN60950 standard.

Electrical connection : The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as «hazardous voltage» according to EN60950 standard).

The apparatus does not incorporate an integral power on/off switch. To disconnect the power, either switch off supply at the mains power socket or unplug the AC adaptor. When installing the apparatus, ensure that the mains power socket is readily accessible.

Telephone connection : Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.

For product support and help
visit our website
at www.geemarc.com
telephone 01707 384438
or fax 01707 372529

ALARM OPERATION

1 - SETTING THE ALARM MODE

Choose which alarm mode you want to be by moving the alarm mode switch to one of the following positions:

0 : no alarm

: both strobe flasher and sound alarm activated

: both strobe flasher and shaker (if connected) activated

: both sound alarm and shaker (if connected) activated

2 - VOLUME CONTROL SLIDE SWITCH

Set the volume of the alarm sound.

3 - TONE CONTROL SWITCH (3 POSITIONS)

Set the tone of the alarm sound.

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantees it for the period of one year. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at www.geemarc.com. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM

Please note: The guarantee applies to the United Kingdom only.

Declaration : Hereby Geemarc Telecom SA declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC and in particular article 3 section 1a, 1b and section 3.

Recycling directives : The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin.

Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.

- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

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