THUASNE

LIGAFLEX EVOLUTION®

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BIAXIAL LIGAMENT KNEE BRACE WITH REINFORCEMENTS

(ENG)

Properties :

- Two-way elastic fabric.
- Lateral reinforcements with biaxial hinge. (1)
- Central support strap for the lateral reinforcements. (2)
- Upper and lower double adjustable strap (anti-tourniquet effect). (3)
- Wide opening for comfort of the popliteal fossa. (4)
- Patellar opening with a silicone guide. (5)

Indications :

- Orthopedic treatment of simple or complicated sprains.
- Post-surgical treatment : convalescence and rehabilitation phases.
- Functional compression in the case of hyperlaxity.

Precautions :

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs.

Store at room temperature, preferably in its original packaging.

Re-use of this device for the treatment of another patient is strictly not recommended due to hygiene- and performance-related problems. If a decision is taken to re-use this device, then the person taking the decision is solely responsible and the Thuasne warranty will automatically become void.

Washing instructions :

- It is preferable to remove the lateral reinforcements before washing. To replace them, bend the knee brace.
- Wash only with soapy water, without detergent or bleach.
- Squeeze out excess water.
- Dry away from heat sources.

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