

User guide for TRP-Platform

Before use

- Please remove all packaging and ensure the product has not been damaged in transit.
- Carefully read the instructions and warning texts before using the product
- Keep instructions for reference
- On each product you will find a label that shows the maximum permitted load.


Contents

- 1 x platform (fig.3), 1 x frame (fig.4), 4 x large legs, 4 x small legs, 2 x raised edges with handles (fig 5), platform/ frame fixings (fig.1), leg fixings (fig.2), 16 x washers (fig.1)

Fig.1



Fig. 2

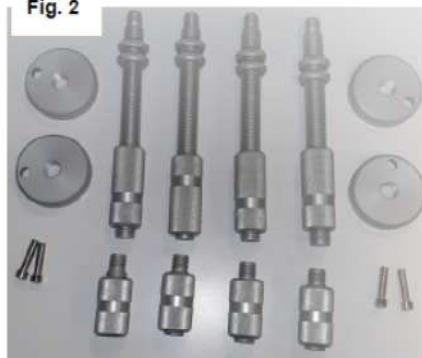

To Assemble

1. Remove handles from raised edges (fig. 6) and reattach handles to raised edges connecting this to the platform using the handle fixings (fig. 7)
2. Once handles and edges are attached to platform, position frame inside the platform (fig. 8)
3. Fix frame in to platform using longer fixings and washers (fig.9 and fig.10)
4. Insert all fixings loosely until all are in the correct position before tightening (fig. 11)
5. Screw legs in to feet of the leg fixings (fig. 12)
6. Set adjustable leg to height required, there are three height options (fig. 13)
7. Insert legs in to the platform and tighten with fixings supplied (fig. 14)

Fig. 3

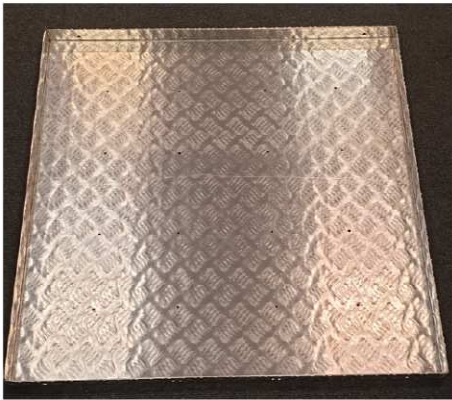


Fig. 4

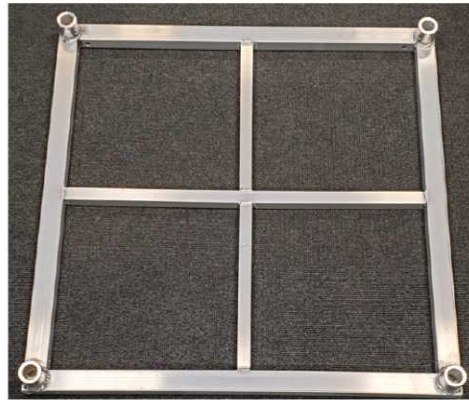


Fig. 5



Fig. 6



Fig. 7



Fig. 8

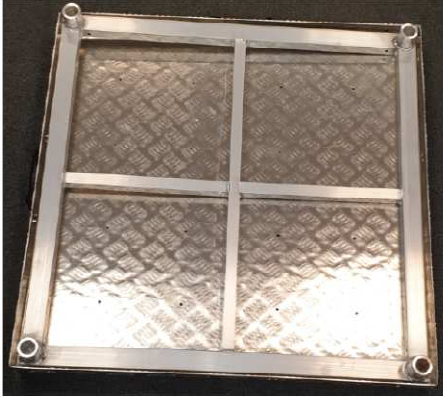


Fig.9



Fig.10



Fig.11



Fig.12



Fig. 13



Fig. 14

