GEEMARC  
**CLEAR SOUND**<sup>®</sup>  
 HEAR THE DIFFERENCE

**GB** User guide p.2

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**CAUTION:** THIS TELEPHONE PRODUCT IS SPECIFICALLY DESIGNED FOR PEOPLE WITH HEARING DIFFICULTIES AND IS PROVIDED WITH A HANDSET EARPIECE VOLUME CONTROL FOR INDIVIDUAL REQUIREMENTS. DUE CARE MUST BE TAKEN BY ALL USERS THAT THE HANDSET EARPIECE VOLUME CONTROL IS SET TO THE LOWEST LEVEL ACCEPTABLE BY EACH RESPECTIVE USER. CARE SHOULD THEREFORE BE EXERCISED TO ENSURE THAT ANY NAIVE USERS ARE FAMILIAR WITH THIS REQUIREMENT.

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<b>Description</b>	<b>4</b>
<b>Setting up</b>	
- Connecting the telephone	6
- Ringer settings	6
- Pulse/Tone switch	7
- Time break recall switch	8
- Wall mounting	8
<b>Using the phone</b>	
- Answering incoming call	9
- Volume and tone receiving adjustment	9
- Making a call	11
- Sending volume adjustment	11
- Last number redial	12
- Recall, * & # buttons	12
- Mute button	12
- Storing telephone numbers	13
- Dialling stored number	14
<b>Guarantee</b>	<b>15</b>

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**Garantie 27**

# DESCRIPTION

Tone and volume adjustment automatically activated each time you hang-up the phone

3 emergency numbers

Tone reception adjustment

Clip for wall mounting

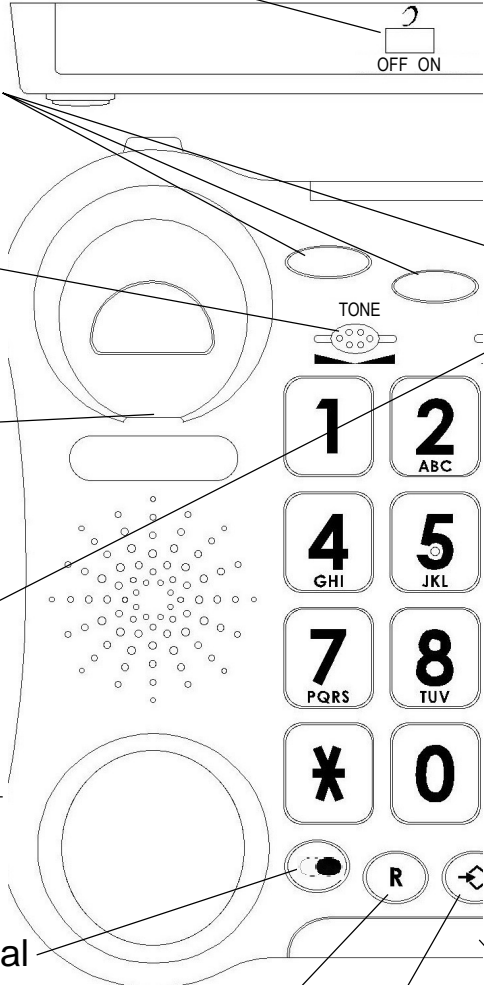
Volume reception adjustment

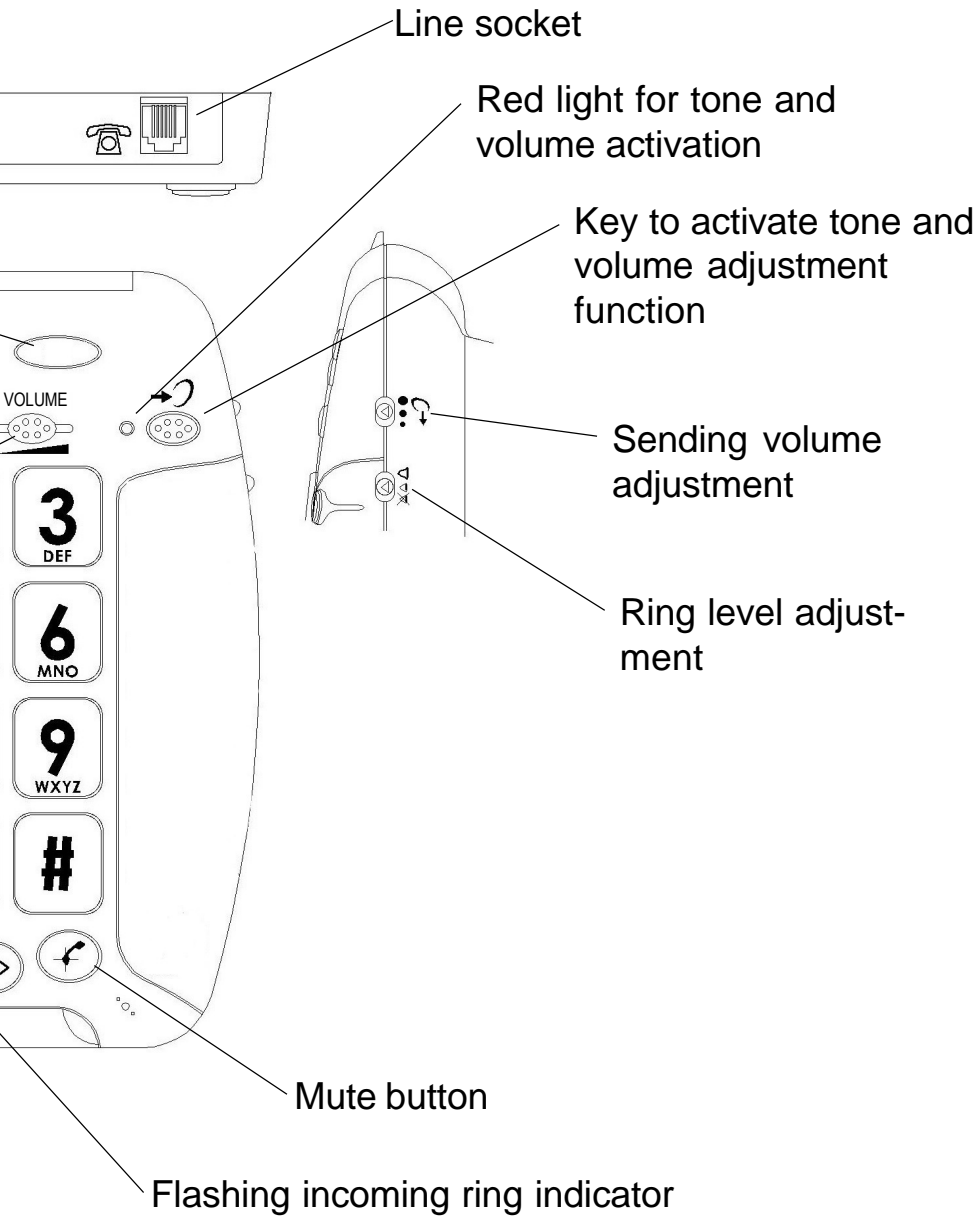
Handset cord socket

Last number redial


Time break recall button

Memory storing key

# SETTING UP

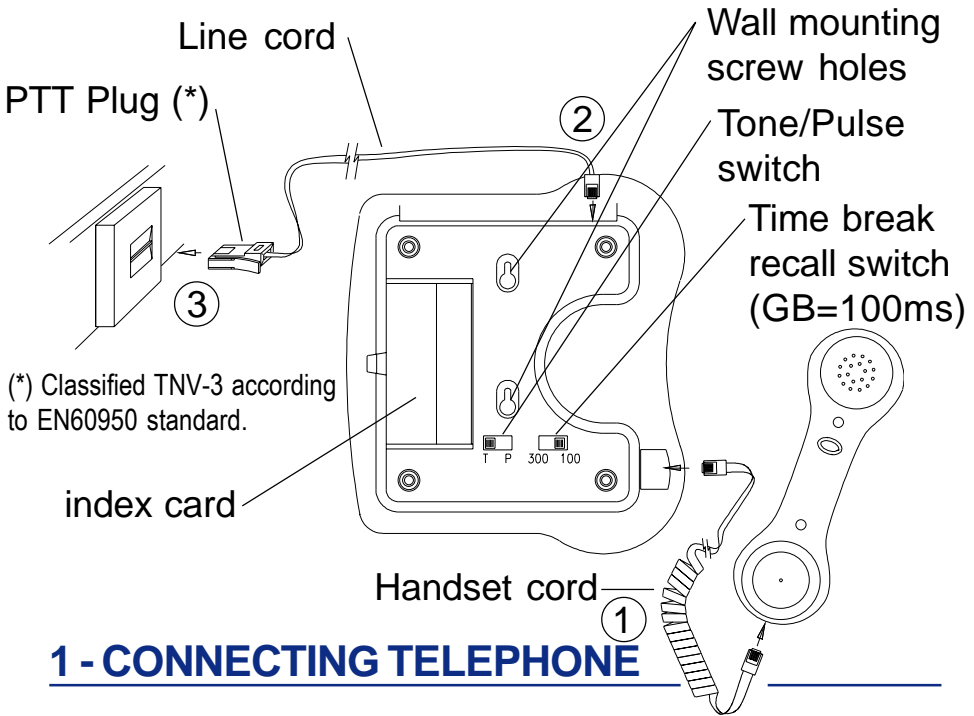- 1 - Connect the handset cord.
- 2 - Connect the line cord.
- 3 - Connect the PTT plug in the wall socket.

## 2 - RINGER SETTINGS

A switch located on right side allows the ringer to be switched HI,LOW,OFF:

 : Off,  : Low,  : Hi

# SETTING UP

Your phone comes with the ringer switched to «High» and the melody set to «6»:

	Low	Mid	High
Melody 1	1	2	3
Melody 2	4	5	6(by default)
Melody 3	7	8	9

If you don't like the ringer melody, you can choose a new one by changing the settings as follow:

- Lift the handset.
- Press  button.
- Press # button.
- Press any button 1 to 9.

Ringer will be activated on next call.

## 3 - TONE/PULSE SWITCH (T/P)

The Tone/Pulse switch is factory pre-set to Tone position. In the UK, all telephone exchanges now use Tone dialing. If your phone does not dial out it is probably being used from an older private switch board (PBX). In this case, the Tone/Pulse switch must be moved to Pulse.

# SETTING UP

## 4 - TIME BREAK RECALL SWITCH (300/100)

For UK, the TBR switch is factory pre-set to 100ms. It can be switched to 300ms for other locations or PBX's.

## 5 - WALL MOUNTING

To wall-mount your telephone, slide out the clip turn around and slide back into the wall mounting position (located in the base unit, just below the ear piece). This will ensure that the handset stays firmly in position. Make 2 holes in the wall 80mm apart, insert the wall plugs and the wall mounting screws. The screws should protrude from wall by 6-7 mm. Place the phone onto the screw-heads and slide down to secure.

***If you are using this telephone with a hearing aid, set your hearing aid to the "T" setting.***

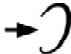
# USING THE PHONE


## 1 - ANSWERING INCOMING CALL

- When an incoming call is received the phone will ring and flash, to answer the call lift the handset and speak.
- On completion of the call, carefully replace the handset in the cradle to release the line.

## 2-VOLUME AND TONE RECEIVING ADJUSTMENT


You can increase the volume and adjust the receiving tone according to your own hearing requirements.

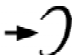The button  allows extra-receiving and tone functions to be activated or not during the conversation.

When the function is activated, the red light of extra-receiving and tone function located near the button  is lit.

# USING THE PHONE

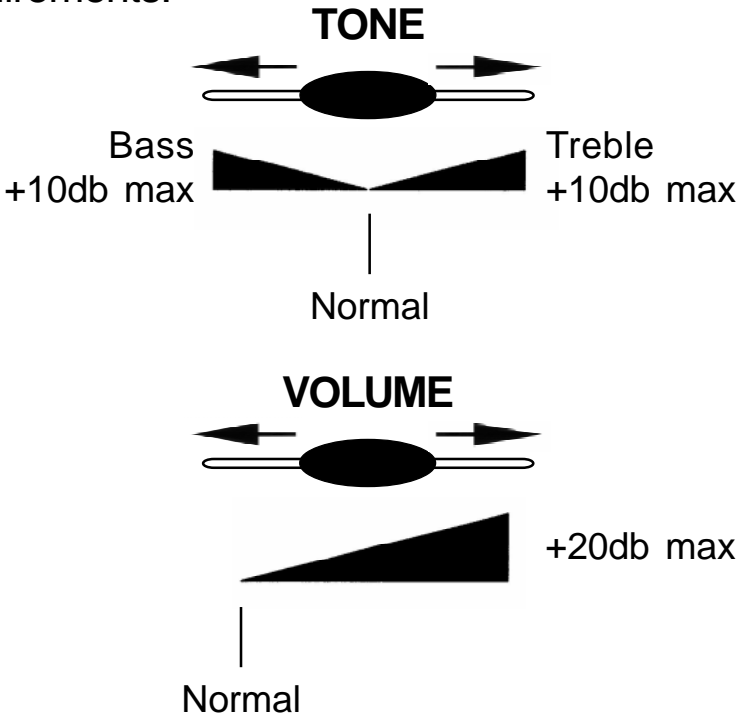Adjust the **TONE** and **VOLUME** buttons to adjust the levels according to your own hearing requirements.

Note :

When the switch  located behind the phone is **ON**, the extra-receiving and tone functions are automatically activated each time you use the phone. When this switch is switched **OFF**, you will need to activate the function by pressing the button .

# USING THE PHONE

## 3 - MAKING A CALL

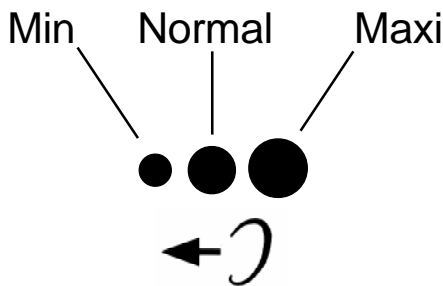
---

- Lift the handset.
- Await the dial tone and dial required number.
- On completion of the call, carefully replace the handset in the cradle.

## 4 - SENDING VOLUME ADJUSTMENT

---

If your voice is quiet or called parties have difficulty hearing you, you can increase the sending level volume with the switch located on the right side. You can also decrease the sending volume if required.

# USING THE PHONE

## 5 - LAST NUMBER REDIAL

---

- Lift the handset.
- Await the dial tone and press  button.
- The previously dialled number will be automatically redialled (not in use for memory numbers).

## 6 - RECALL, \* & # BUTTONS

---

These are used with the new services provided on digital exchanges. For details please contact your network operator.

## 7 - MUTE BUTTON

---

If you wish to speak privately to someone else in the room without your caller overhearing, press and hold the  button. You will still be able to hear the callers but they cannot hear you during the mute operation. To resume normal conversation, release the  button.

# USING THE PHONE

## 8 - STORING TELEPHONE NUMBERS

Your phone can store 13 memories including 3 direct priority numbers (P1, P2 or P3) (21 digits maximum for each memory).

For example, the 3 direct priority numbers allow you to store the 3 numbers below:

- Fire, Police,  
Ambulance:999
- Your doctor

*It is recommended to store this numbers during installation*

To store a number please follow instructions below:

- Lift the handset.
- Press → button.
- Press **P1**, **P2** or **P3**.

Or press **MEM** and any button **0** to **9** on the keypad under which you wish to store the telephone number.

- Dial the number you wish to store.
- Press → button again.

Numbers will be lost after few hours if you disconnect your phone. Entering a new number in memory will automatically erase the previous number.

# USING THE PHONE

## 9 - DIALLING STORED NUMBER

---

- Lift the handset and await the dial tone.

- Press appropriate button **P1**, **P2** or **P3**.

Or press **MEM** button followed by the keypad **0** to **9** button under which the desired number is stored.

The stored number will be dialled automatically.

To register your stored phones numbers, use the index located on the base.

Push the plastic window on the right to write on the index.

Carefully replace the window on the base.

# GUARANTEE

## GUARANTEE

---

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at [www.geemarc.com](http://www.geemarc.com). The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM**

**Please note:** The guarantee applies to the United Kingdom only.

**Declaration :** Hereby Geemarc Telecom SA declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC and in particular article 3 section 1a, 1b and section 3.

**Telephone connection :** Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.

The telephone does not operate if the line current is lower than 18 ma.

### **Recycling directives**

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin.

Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.

- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

**For product support and help visit our website**

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